



Access Your World

# Pressalit Care 3000 Changing Table

## Important safety information

### Do not leave occupant unattended.

- Please read this manual thoroughly. We take no responsibility for any unspecified uses.
- The information in this manual pertains to correctly installed equipment.
- The product is not intended to be played with.
- Do not use the changing table if it is out-of-order or damaged.
- This manual is subject to alteration without notice.
- Make sure that nothing interferes with the table when in motion.
- Children should not operate the changing table unsupervised.
- Verify voltage compatibility before initial operation.
- Support the table while unfolding to avoid damage.

## Using the changing table

- To use either end of the bench mattress as a head/neck support, pull the section out of the mattress frame. Adjust the height by positioning the metal support struts.
- To position the safety rail, push down against the rail to unlock it from the underside of the bench. Swing the rail up and towards you. It locks into position automatically.
- To fold the safety rail out of the way, the two locks must be released at the same time. Swing the rail towards the underside of the bench. Push the rail against the underside of the bench to lock into place.

## Height adjustment *(height adjustable models only)*

- Adjust the height of the bench by using the indicators on the hand control. Press and hold the 'up' or 'down' button to initiate. Release to stop the movement.
- When mounted at the recommended height, the bench can be adjusted from 12" (300 mm) to 34" (1000 mm) above the floor.

If the changing table stops functioning, check the power, wiring, and plugs. If there is something wrong or suspicious, please contact a qualified electrician or MAX-Ability for further assistance.



## Cleaning

All components are water compatible. Remove the mattress sections to clean the water collection tray. Most general purpose cleaners and disinfectants can be used. Avoid harsh brushes, abrasives, undiluted bleach products, or cleaning agents containing acids or chloride. Rinse off with clear water and dry with a soft cloth. Clean after every use.

Occasionally, remove limescale marks on aluminium parts. Use 3 parts 30% household vinegar to 7 parts water. Rinse with a cloth wrung out in clean water

## Maintenance

*(Height-adjustable models)* Fully activate the motor at least once every week by raising the product to its highest level and lowering it to its lowest level in order to lubricate the internal parts.

Inspect the changing table completely each year (with a licensed contractor if necessary):

- Remove the upper and lower securing screws of the coach housing and carefully remove the housing.
- Check wall mounting fixtures and tighten if required.
- Check all moving parts for wear.
- Check all mountings on nursing bench, safety bar and securing brackets and tighten as required.
- Check waste connections and outlet pipe-work, check and clean trap (where fitted).
- Check all electrical connections to the control box.
- Check isolation fuse (5 Amp) and change if incorrect.
- Test operation of all systems.
- Carefully refit the coach housing and secure.
- Check mattress sections for any damage and clean as recommended.
- Report of any defects, damage, or missing parts.

Verify your purchase details before carrying out repairs so you don't void your warranty:

*We guarantee all product components against manufacturing defects for three years, and parts repair/labor for one year, from the invoice date. In most instances the affected parts are replaced, not repaired.*

*Parts damaged due to improper product assembly, installation, or procedure; lack of recommended maintenance including cleaning; or lost after delivery cannot be included in this warranty.*

For any questions or problems, please contact us at 1 800 577-1555 or [www.max-ability.com](http://www.max-ability.com).

Subject to change without further notice