

Pressalit Care LS-BA Koala Changing

Table User Manual

(Models R8661/R8662/R8663)

Important safety information

Do not leave occupant unattended.

- Please read this manual thoroughly. We take no responsibility for any unspecified uses.
- The information in this manual pertains to correctly installed equipment.
- The product is not intended to be played with.
- Do not use the changing table if it is out-of-order or damaged.
- This manual is subject to alteration without notice.
- Make sure that nothing interferes with the table when in motion.
- Children should not operate the changing table unsupervised.
- Verify voltage compatibility before initial operation.

Using the changing table

- To position the safety rail, release the lock and pull from the underside of the bench. Swing the rail up and towards you. It locks into position automatically.
- To fold the safety rail out of the way, release the lock and swing the rail towards the underside of the bench. Push the rail against the underside of the bench to lock into place.

Height adjustment

- Adjust the height of the bench by using the indicators on the hand control. Press and hold the 'up' or 'down' button to initiate. Release to stop the movement.

If the changing table stops functioning, check the power, wiring, and plugs. If there is something wrong or suspicious, please contact a qualified electrician or MAX-Ability for further assistance.



Cleaning

All components are water compatible. Most general purpose cleaners and disinfectants can be used. Avoid harsh brushes, abrasives, undiluted bleach products, or cleaning agents containing acids or chloride. Rinse off with clear water and dry with a soft cloth. Clean after every use.

Occasionally, remove limescale marks on aluminium parts. Use 3 parts 30% household vinegar to 7 parts water. Rinse with a cloth wrung out in clean water

Maintenance

Fully activate the motor at least once every week by raising the product to its highest level and lowering it to its lowest level in order to lubricate the internal parts.

Inspect the changing table completely each year (with a licensed contractor if necessary):

- Remove the upper and lower securing screws of the coach housing and carefully remove the housing.
- Check all moving parts for wear.
- Check all mountings on changing table, safety rail and securing brackets and tighten as required.
- Check all electrical connections to the control box.
- Check isolation fuse (5 Amp) and change if incorrect.
- Test operation of all systems.
- Carefully refit the coach housing and secure.
- Report of any defects, damage, or missing parts.

Verify your purchase details before carrying out repairs so you don't void your warranty:

We guarantee all product components against manufacturing defects for three years, and parts repair/labor for one year, from the invoice date. In most instances the affected parts are replaced, not repaired.

Parts damaged due to improper product assembly, installation, or procedure; lack of recommended maintenance including cleaning; or lost after delivery cannot be included in this warranty.

For any questions or problems, please contact us at 1 800 577-1555 or www.max-ability.com.

Subject to change without further notice