R8574572/R8575572  
R8584572/R8585572  
R8594572/R8595572

## CT 4000/CT 4100

**Change table – Operation and maintenance manual** en

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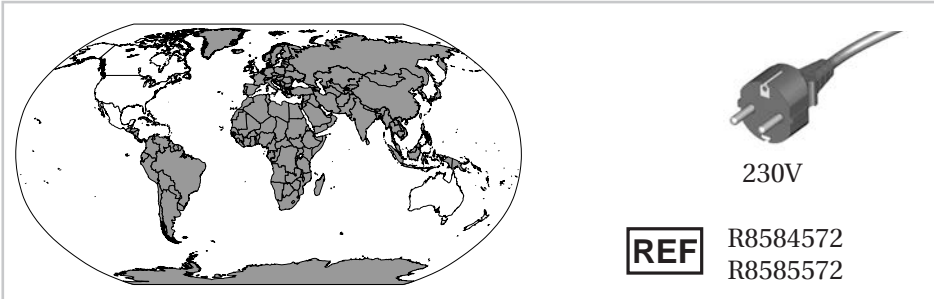
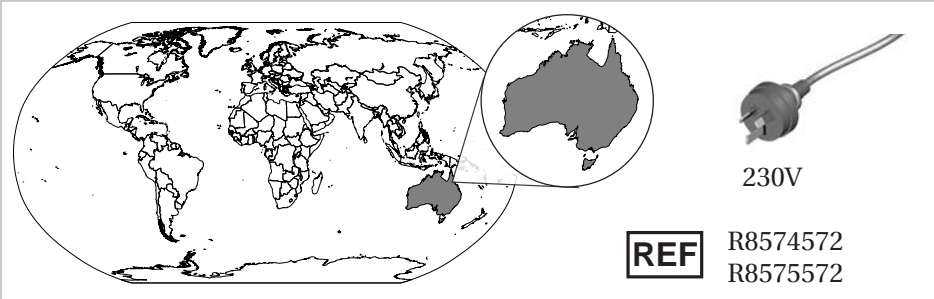
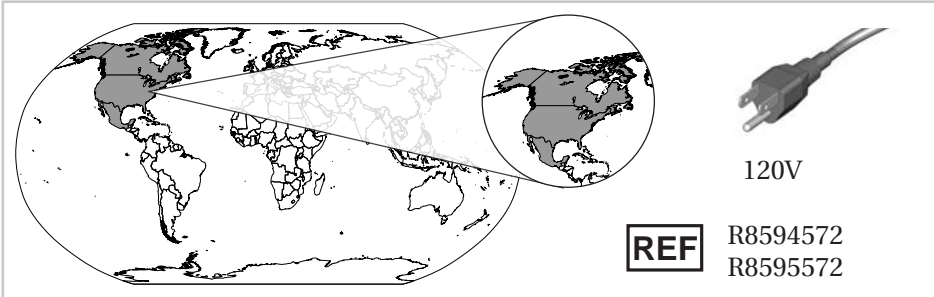
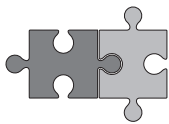
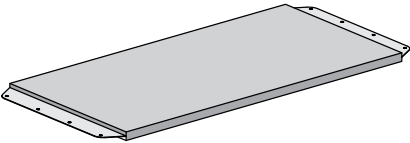
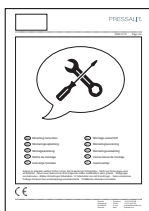
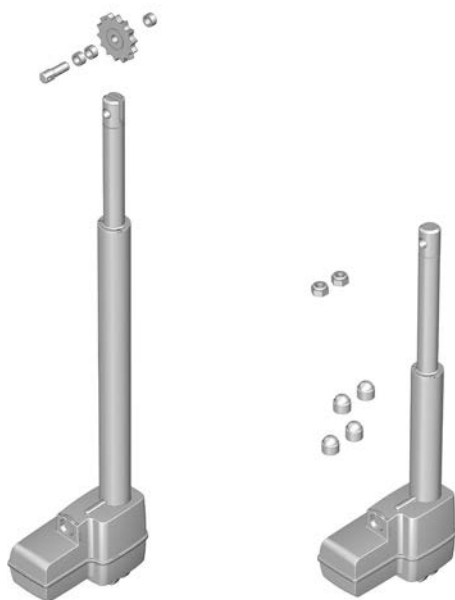
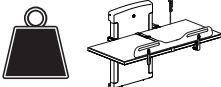
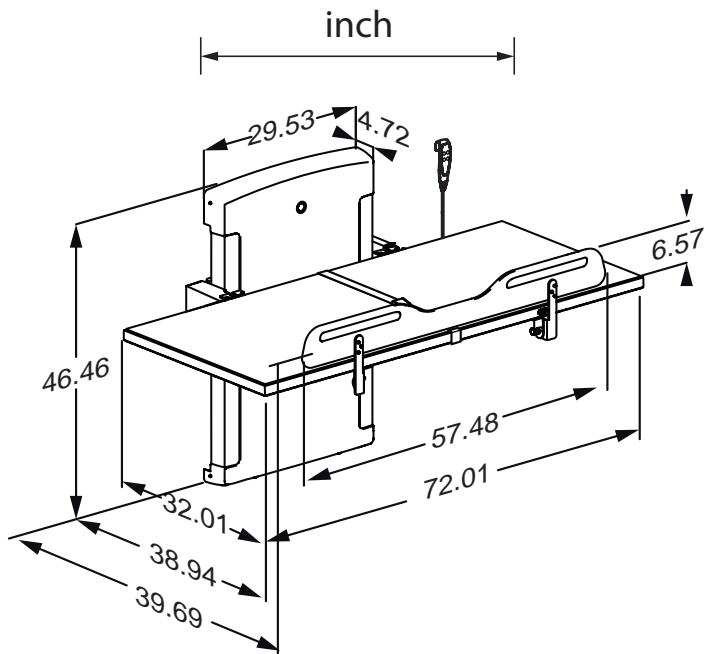
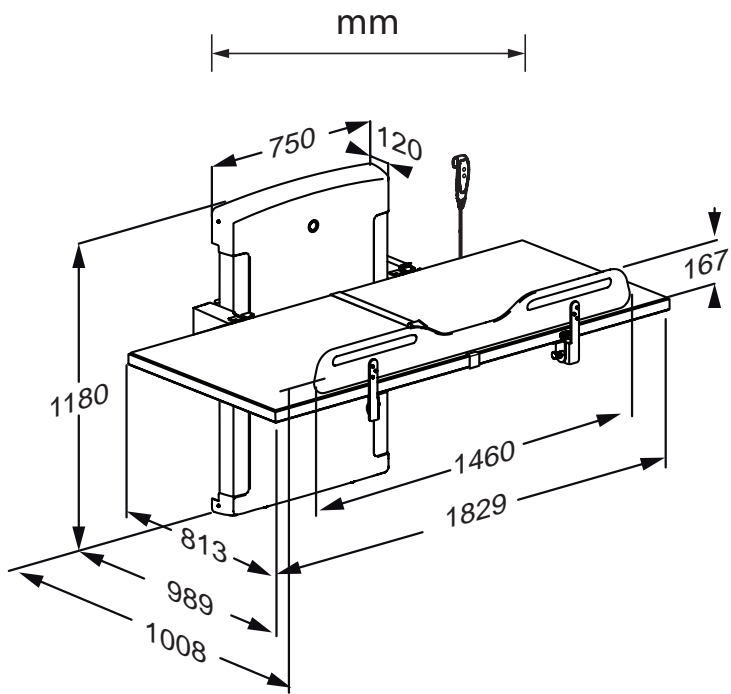
en

## Mounting instruction

A mounting instruction is included with the product. It can also be downloaded from [pressalit.com](http://pressalit.com).

## Operational signage

An operational signage is included with this product. It can also be downloaded from [pressalit.com](http://pressalit.com).

Original instructions

# Symbols used in this manual

The following symbols are used in this manual as well as the associated documentation supplied with the product.

 <b>WARNING</b>		<p><b>WARNING</b> indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.</p>	
 <b>CAUTION</b>		<p><b>CAUTION</b> indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices.</p>	
<b>IMPORTANT</b>		<p><b>IMPORTANT</b> indicates important information about handling and use of the product.</p>	
<b>NOTICE</b>		<p><b>NOTICE</b> is used to address practices not related to personal safety.</p>	
 <p>Hand injury warning</p>	 <p>Head injury warning</p>	 <p>Electricity / voltage warning</p>	 <p>Other specific warning</p>
 <p>Compatible products</p>	 <p>Spare parts</p>	 <p>Power specifications</p>	 <p>EMC related symbols</p>
 <p>Product weight</p>	 <p>Max. user weight</p>	 <p>Article number / versions</p>	 <p>Do not leave patient unattended</p>

en

## General safety

### **WARNING**

- Read and understand the manual completely before use. Keep the manual close to the unit.
- Comply with this manual to avoid accidents involving serious personal injury.
- Never use or handle this product in other ways than specified in this manual, as this can put your safety at risk and cause damage to the product.
- Make sure that everyone who is to install or use the product and its systems has the necessary information and access to this manual.

### **CAUTION**

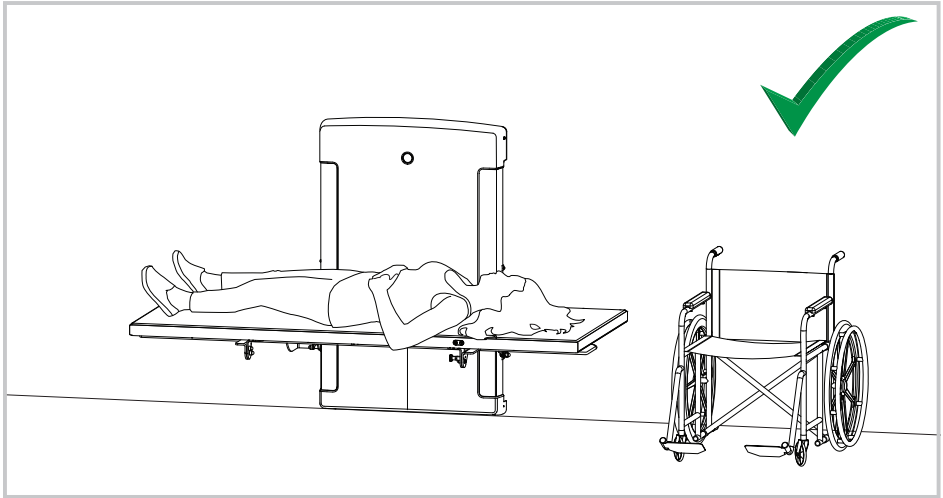
- Please comply with these instructions to avoid the product becoming damaged or destroyed.
- Do not perform repairs, disassembly, or assembly operations, extensions, re-adjustments or modifications to this product. These must be carried out by Pressalit or by persons authorized by Pressalit only.
- Never allow children to operate this product unless under supervision. The product is not intended for play.
- Never use the product if it is defective or damaged. Do not use a defective product before it has been repaired.

### **NOTICE**

- The information in this manual is based on correct fitting of the product in accordance with our mounting instruction.
- Pressalit cannot be held liable if the product is used in any way which differs from that described in the guide.
- We reserve the right to amend this manual without prior notice.

# Intended use


## Using alone (no helper)


 **CAUTION**

en

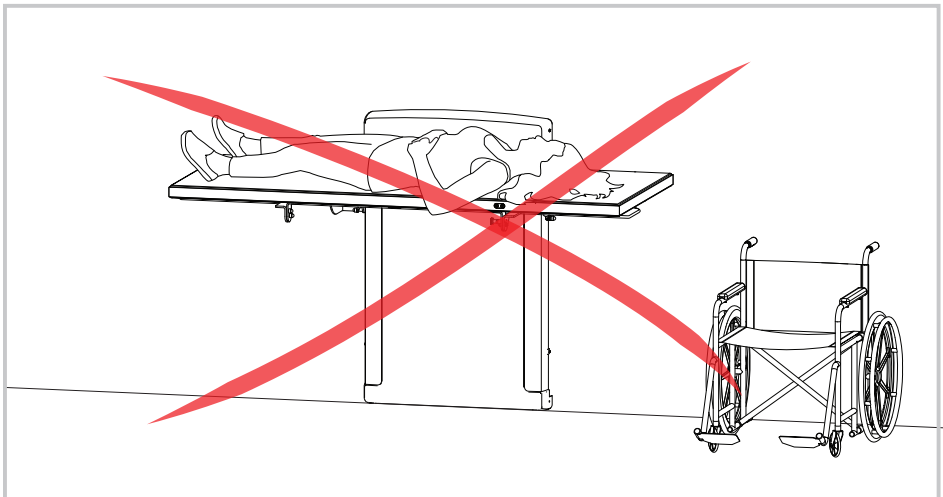This product has no manual lowering function, so in case of a power outage, the lying surface will stay in its current position. It is therefore not recommended to take the lying surface to the highest position when using the product alone.


## Using with a helper.

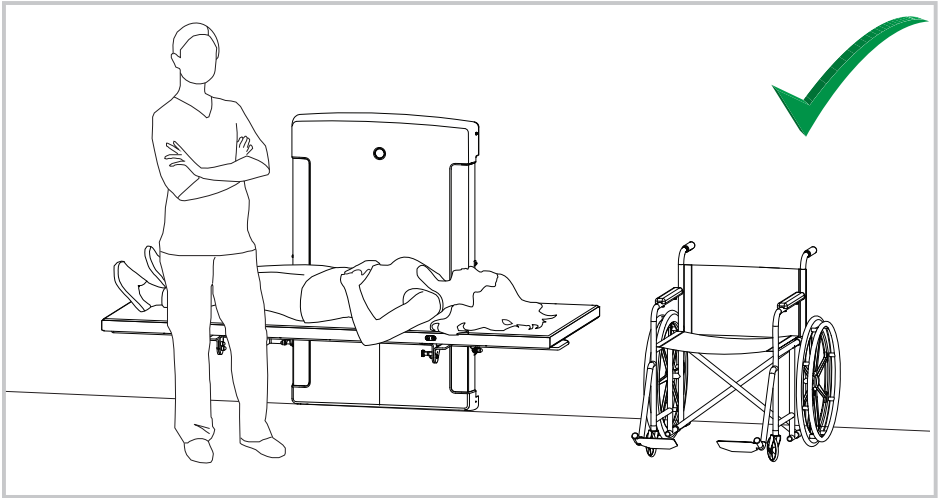en

 **CAUTION**

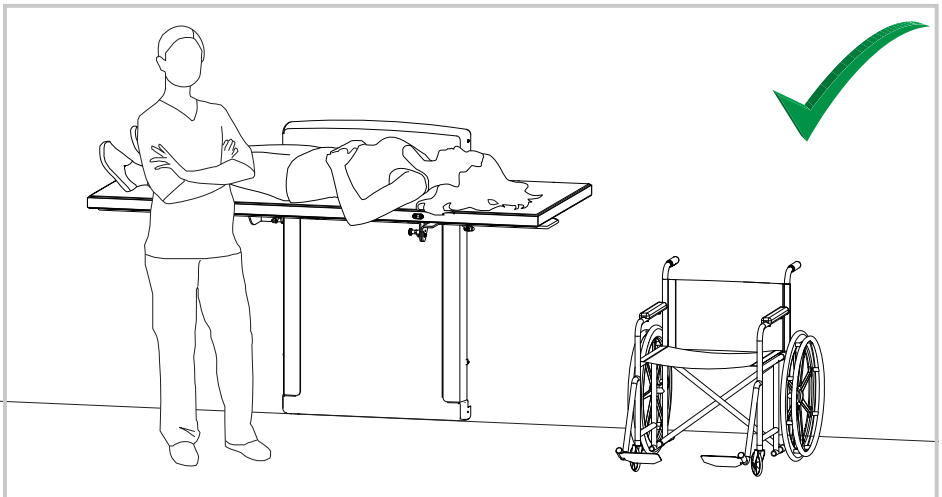It is only recommended to take the lying surface to its highest position when using this product in the presence of a helper.

This product is intended for people who

- need to get changed on a change table
- are above the usual change table age
- are caregivers and need a customized work position
- need to use a change table in combination with person lifts

### **Intended operator profile**

The product is to be used by a person trained in health care and/or trained in changing a person in their care. It can also be used by self-reliant users.

### **Intended user profile**

The product can be used by adults whose weight does not exceed the **maximum load of 227 kg / 500 lbs.**

### **Operating environment**

For indoor use in public changing facilities and schools. Can also be used in nursing homes and hospitals under normal working conditions.

### **Contraindications for use**

Use of the change table is not recommended for patients with infusion or insulin pump, inflammatory skin diseases or patients in feverish conditions.

Patients with cardiac insufficiency, cardiovascular diseases, craniocerebral injury, open wounds, feed or drain lines (e.g. catheter, drainage tube) may use the change table only after consulting with the attending physician. Do not use if the patient's weight exceeds 227 kg / 500 lbs.

### **Biocompatibility**

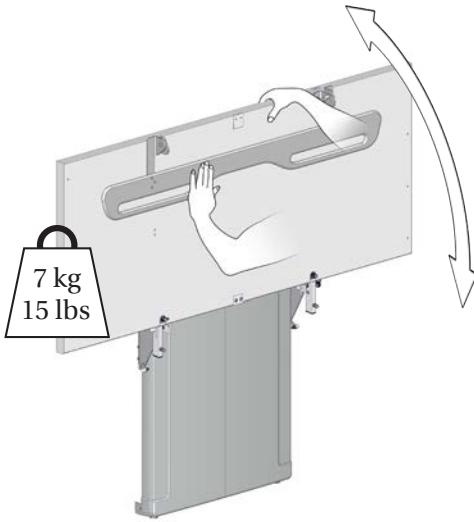
The materials in contact with the skin of the user during the intended use have been evaluated for their content of harmful substances according to the procedures in BS EN ISO 10993-1:2020 Biological evaluation of medical devices – Part 1 and are therefore considered suitable for this purpose.

- Never allow children to operate the unit without supervision of an adult.
- Keep pets and pests away from the unit.

# Operating instructions

## Folding up and down, CT 4000

Support the table using your hand or body when folding it down, to avoid the risk of damage.

The built-in gas cylinders make it easy to fold the table up against the wall after use. Be sure to fold the safety rail away first.

en

**Do not leave the patient unattended.**

 **CAUTION**

Always be careful when moving a person to and from the table.

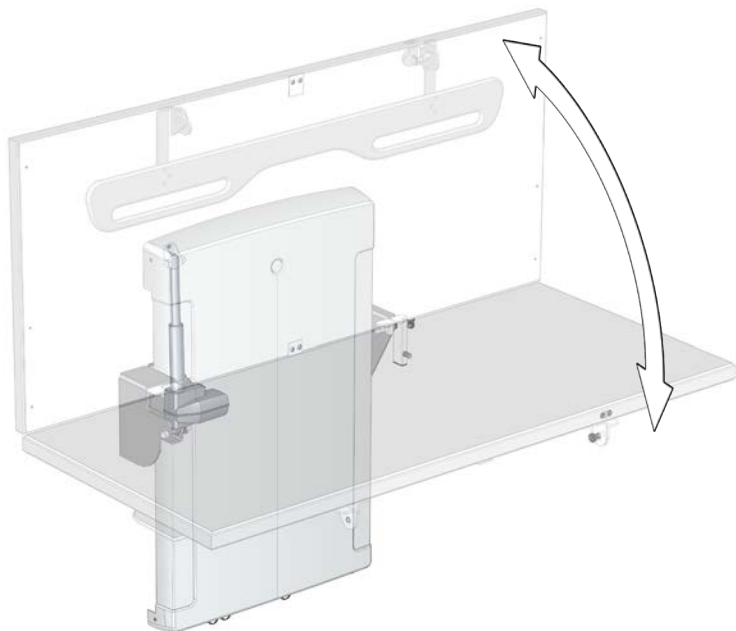
**Take care to avoid injury to hands, feet and head when in the vicinity of moving mechanical parts.**

**NOTICE**

Any serious incident that has occurred in relation to the product should be reported to the manufacturer and the health authorities.

## Folding up and down, CT 4100

en

The table folds up and down electrically

On the wired remote control for this model, there are two additional buttons that control the fold-up function.

Press and hold the fold up button ② to fold the table up, and press and hold the fold down button ③ to fold the table down. The movement stops when the button is released.


**Take care to avoid injury to hands, feet and head when in the vicinity of moving mechanical parts.**
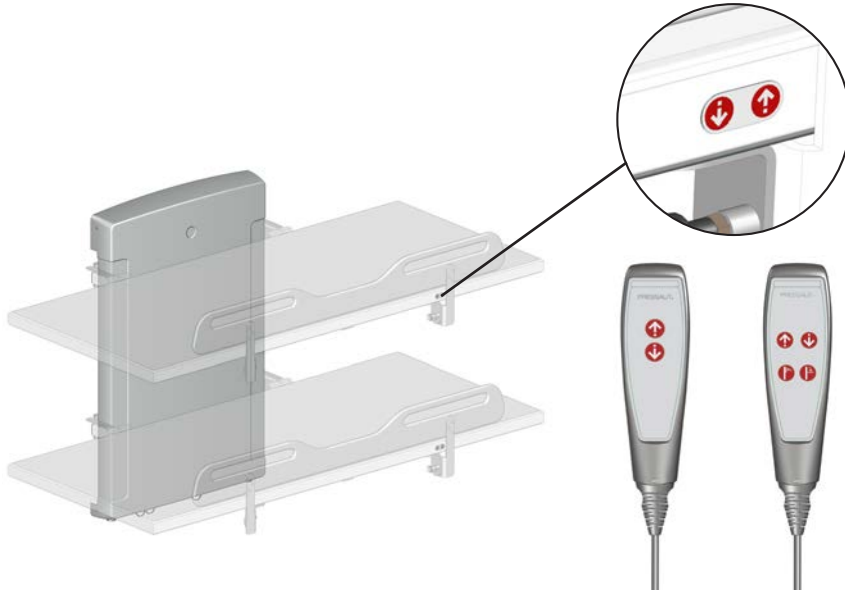
## Height adjustment

When the change table is installed at the recommended height, it can be adjusted between 300 and 1000 mm (11.81 and 39.37 inches) above the floor.

The height adjustment is done by pressing the buttons on the frame or on the wired remote control unit.

en

Press and hold the Up button to raise the table, and press and hold the Down button to lower the table. The movement stops when the button is released.

The height adjustment may only be used at temperatures between 15 and 30 degrees Celsius.


**Make sure that there are no objects under the change table when adjusting the height.**

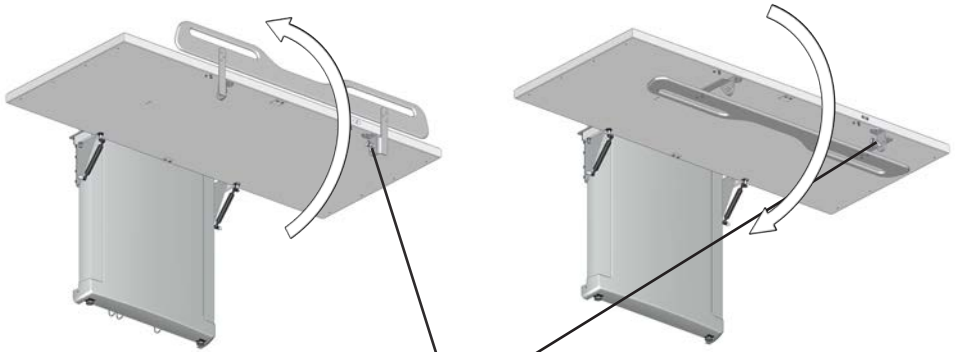**To avoid the risk of strangulation, make sure that the wired remote control is placed in its holder when not in use.**

## The safety rail


**To pull the safety rail up:** Release the lock ① and pull the safety rail from the underside of the table, and swing it upwards. The safety rail will automatically lock into position.

**To fold the safety rail away:** Release the lock ① and swing the safety rail down to the underside of the table. The safety rail will automatically lock into position.

en

## The body belt

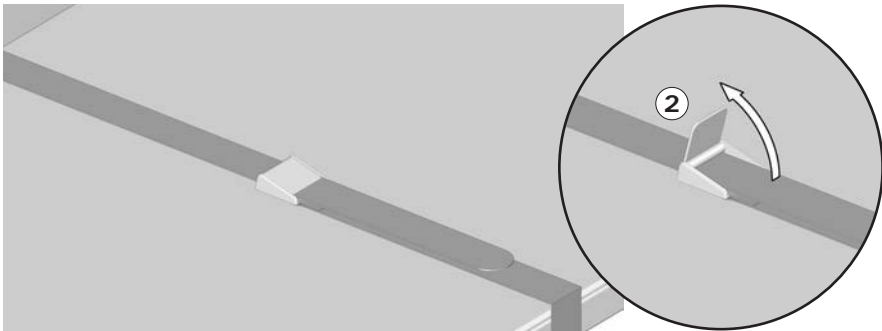The product is equipped with a body belt, that can be used to secure the user. Pull the belt lock lever ② up to release the belt lock. Press the lever down again to lock the body belt.

## Cleaning

The product is to be cleaned with hot water or a mild soapy cleaning agent. To see which cleaning products have been tested and approved by Pressalit for the specific product, please visit [Pressalit.com](http://Pressalit.com) and refer to the product's item number. Pressalit recommends cleaning the product after installation.

### CAUTION

Always read the material safety data sheet (MSDS) and supplied instructions of the cleaning, disinfecting or de-scaling agent and comply with instructions for safe use, dilution, rinsing, drying, handling and storage of the agent.

### IMPORTANT

- Do not use an autoclave or other type of steam cleaner.
- Avoid aggressive cleaning agents containing acids.
- Never spray the product directly with a high pressure cleaner.
- Avoid use of abrasive, corrosive or chlorine-based cleaning or disinfecting products.

## Disinfection

When using the change table for multiple patients, it is necessary to both clean and disinfect it between uses on different patients to prevent the risk of cross infection.

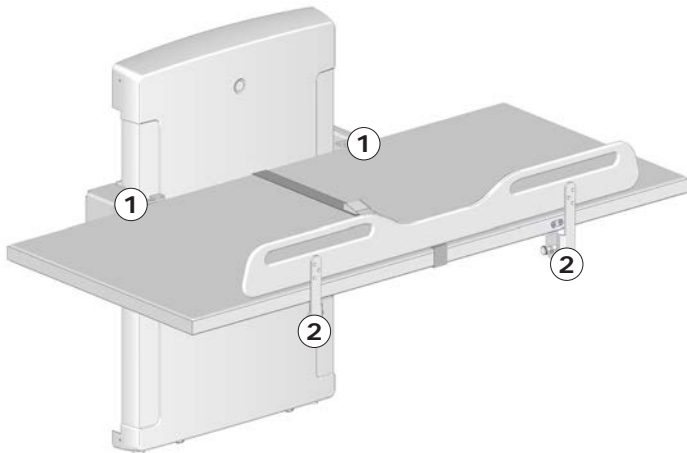
# Maintenance

Expected service life of the change table is 10 years, provided that maintenance is carried out as described below:

It is recommended on a weekly basis to run the motors on the product to their maximum positions by raising the table to its full height, and then lowering it to its lowest position, so that the internal parts are lubricated.

It is recommended that inspection is done once a year.


## Maintenance inspection

Check the hinges and other fittings ① on the table and the safety rail ②. Tighten if necessary.


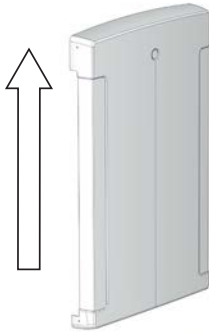
Do not perform any type of maintenance inspection on the change table while a patient is lying on it.

**Disconnect mains before removing the covers.**

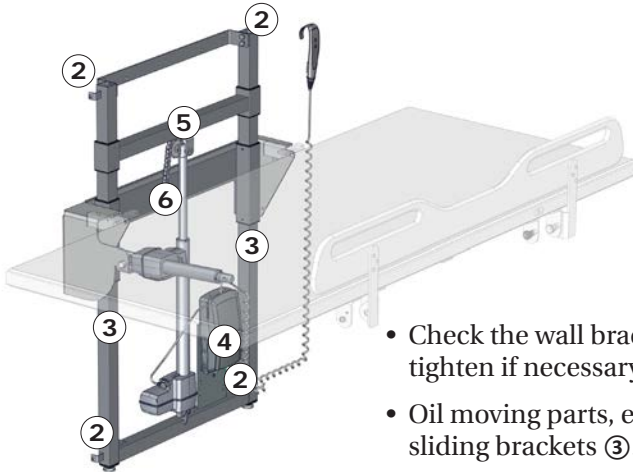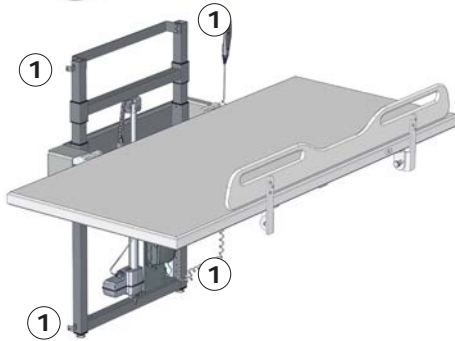
Remove the upper and lower retaining screws ① on the lift unit cover and carefully remove the cover.

- Check the wall brackets ② and tighten if necessary.
- Oil moving parts, especially the sliding brackets ③.

- Check all electrical connections to the control box ④.
- Check the chain wheel ⑤ for wear.
- Oil the chain ⑥.
- Put the unit cover back again and tighten the retaining screws.

# Product label

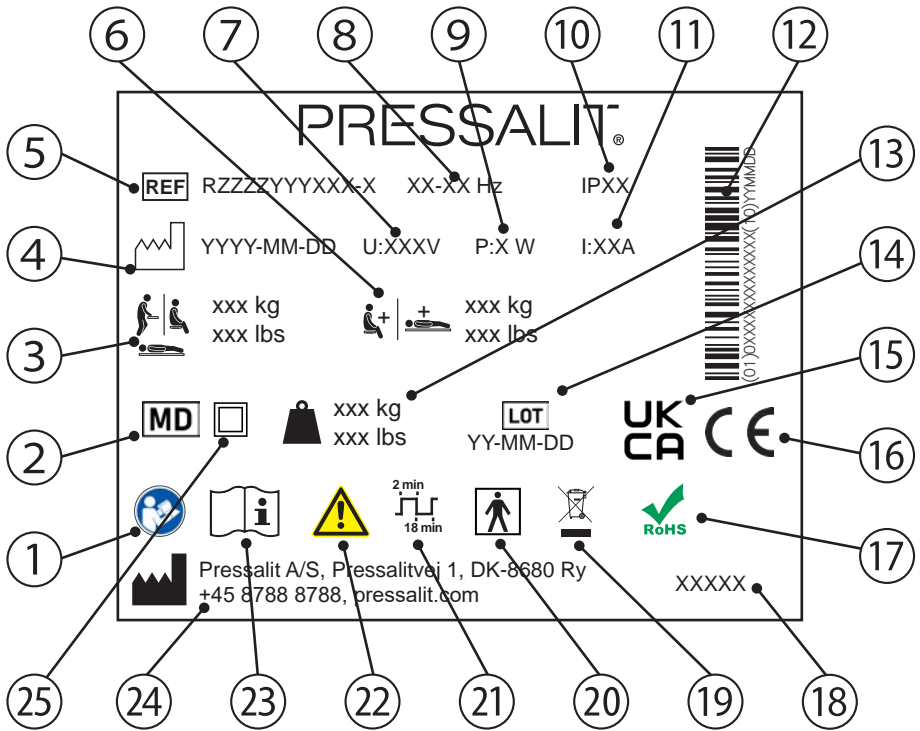en

- ① Consult manual for use

---

- ② Medical device

---

- ③ Maximum user weight

---

- ④ Production date

---

- ⑤ Article number

---

- ⑥ Max load

---

- ⑦ Mains voltage

---

- ⑧ Frequency

---

- ⑨ Maximum power consumption

⑩ IP class

---

⑪ Electric Current

---

⑫ GS1-128 barcode

---

⑬ Product weight

---

⑭ Batch code

---

⑮ UK Conformity Assessed

---

These products are CE-marked in accordance with relevant requirements in:

- ⑯
- Regulation (EU) 2017/745 (5 April 2017) on medical devices.
  - Council Directive 2006/42/EC (of 17 May 2006) on machinery.
  - Council Directive 2011/65/EC (of 8 June 2011) on RoHS.

A declaration of conformity can be found at [pressalit.com](http://pressalit.com)

---

⑰ This product does not contain any of the hazardous substances mentioned in the Council Directive 2011/65/EC on RoHS.

---

⑱ Product name (if any)

---

⑲ Do not dispose as unsorted municipal waste. The product should be collected separately and returned to the designated recycling service.

---

⑳ Type BF applied part Part of the product in normal use necessarily comes into physical contact with the patient to perform its function  
The product complies with requirements of IEC 60601-1 to provide protection against electric shock

---

㉑ Duty cycle

---

㉒ Consult the manual for important cautionary information such as warnings and precautions

---

㉓ Consult manual for use

---

㉔ Name and address of manufacturer

---

㉕ Class II Product, double insulated

---

# Resetting

If the height adjustment does not work, and the control box beeps and/or flashes, the unit must be reset.

Press and hold the up button and the down button simultaneously. The control box emits a beep sound. When the beep stops, release both buttons.

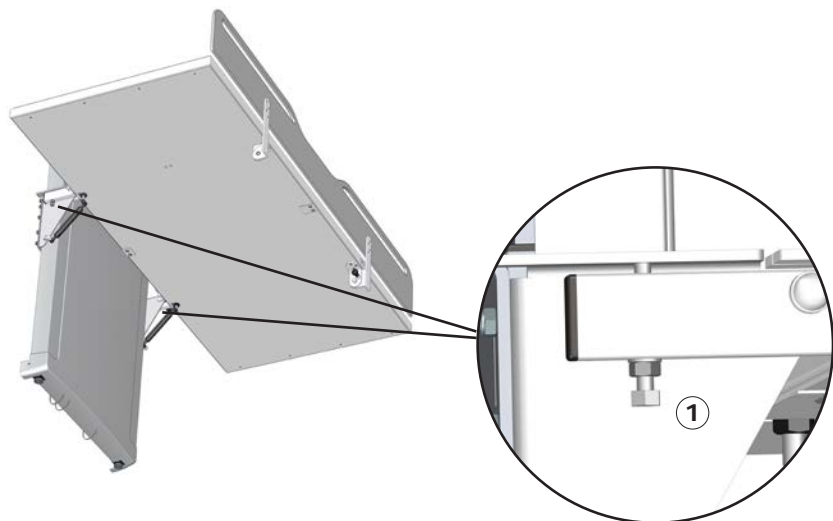
Take the unit to its lowest position. The unit is now reset.

# Trouble-shooting

Symptom	Cause	Action
The unit does not stay in upright position.	One or more gas cylinders are defective.	Contact Pressalit A/S or your local dealer.
The product will not work, even though the remote control is used correctly.	Fault in power supply.	Check that all wires and plugs are intact and correctly connected. Contact Pressalit A/S or your local dealer.
The product does not work, and the control box beeps and/or flashes.	The product needs to be reset.	See "Resetting".

en

The unit runs up and down unevenly.	The sliding brackets are not properly lubricated.	Oil the sliding brackets as described under "Maintenance".
The table top is not level.	Adjustment screws too high or too low.	Adjust the adjustment screws ①.

en

Contact Pressalit or your dealer if the problem cannot be remedied. Do not attempt to repair the product yourself, as this will invalidate the guarantee.

# Electromagnetic compatibility

The device is suitable for use at home, at daycare centres, at day centres for persons with physical or mental disabilities or at hospitals except near active HF surgical equipment and RF shielded rooms with medical systems for magnetic resonance imaging with high intensity of electromagnetic disturbances.

The device does not have any function related to basic safety or essential performance. In case of loss or degradation of performance due to electromagnetic disturbances, the user still can get on and off the product.

<b>Cable</b>	<b>Length</b>
Power cable	3,2 m (126")
Hand control spiral cable	Approx. 5 m (197") extended

- Use of accessories, transducers and cables other than those specified or provided by the manufacturer could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper function.
- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the unit, including its cables, to avoid any degradation of the performance of this unit.
- Use of this equipment adjacent to or stacked with other equipment should be voided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

en

## Electromagnetic immunity

<b>Electrostatic discharge</b>	
	The product must not be affected by electrical discharge between a person and the product in such a way as to initiate actions or render components defective.
<b>Test standard</b>	<b>Immunity test level</b>
EN 61000-4-2	+/- 8 kV contact +/- 2, 4, 8, 15 kV air

<b>Radiated RF electromagnetic field</b>	
	The product must not be affected by radiating RF signals, for example from a mobile phone or radio, that could cause the product to take action or to make components defective.
<b>Test standard</b>	<b>Immunity test level</b>
EN 61000-4-3	10 V/m 80 MHz to 2,7 MHz 80 % AM 1 kHz

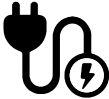
en

**Immunity to proximity fields from RF wireless communication equipment**

	<p>The product must not be affected by radiating RF signals, for example from a mobile phone or radio, that could cause the product to take action or to make components defective.</p>
<p><b>Test standard</b></p>	<p><b>Immunity test level</b></p>
<p>EN 61000-4-3*</p>	<p>385 MHz, 27V/m, 18 Hz PM (50 % duty cycle square wave)            450 MHz, 28V/m, FM +/- 5 kHz deviation, 1 kHz sine            710 MHz, 9V/m, 217 Hz PM (50 % duty cycle square wave)            745 MHz, 9V/m, 217 Hz PM (50 % duty cycle square wave)            780 MHz, 9V/m, 217 Hz PM (50 % duty cycle square wave)            810 MHz, 28V/m, 18 Hz PM (50 % duty cycle square wave)            870 MHz, 28V/m, 18 Hz PM (50 % duty cycle square wave)            930 MHz, 28V/m, 18 Hz PM (50 % duty cycle square wave)            1720 MHz, 28V/m, 217 Hz PM (50 % duty cycle square wave)            1845 MHz, 28V/m, 217 Hz PM (50 % duty cycle square wave)            1970 MHz, 28V/m, 217 Hz PM (50 % duty cycle square wave)            2450 MHz, 28V/m, 217 Hz PM (50 % duty cycle square wave)            5240 MHz, 9V/m, 217 Hz PM (50 % duty cycle square wave)            5500 MHz, 9V/m, 217 Hz PM (50 % duty cycle square wave)            5785 MHz, 9V/m, 217 Hz PM (50 % duty cycle square wave)</p>

en


**Electrical fast transient/burst**

	<p>The product must not be affected by rapid pulse signals through the cord. This could cause the components to become defective, or the product could initiate actions.</p>
<p><b>Test standard</b></p>	<p><b>Immunity test level</b></p>
<p>EN 61000-4-4</p>	<p>Signal line: +/- 1 kV,            AC power line: +/- 2 kV            100 kHz repetition frequency</p>


### Surge immunity

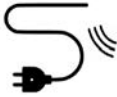	The product must not be affected to initiate actions or to render components defective as a result of lightning.
<b>Test standard</b>	<b>Immunity test level</b>
EN 61000-4-5	Line-to-line: +/- 0.5 kV, +/-1 kV Line-to-ground: +/- 0.5 kV, +/-1 kV, +/- 2 kV

### Immunity to conducted disturbances induced by RF fields

	The wires must not be affected by other RF fields so as to initiate actions or render components defective.
<b>Test standard</b>	<b>Immunity test level</b>
EN 61000-4-6	3 V (6 V in ISM bands and amateur radio bands) 0,15 to 80 MHz , 80 % AM at 1 kHz

### Voltage dips

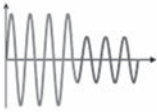	The product must not initiate or limit actions as a result of a dip in the power supply.
<b>Test standard</b>	<b>Immunity test level</b>
EN 61000-4-11	0 % $U_T$ ; 0.5 cycle at 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315°, 0 % $U_T$ ; 1 cycle at 0°, 70 % $U_T$ ; 25/30 cycles at 0° Single phase at 0°

### Voltage interruptions

	The product must not initiate or limit actions as a result of a short-term power supply stop.
<b>Test standard</b>	<b>Immunity test level</b>
EN 61000-4-11	0 % $U_T$ ; 300 cycle. 250/300 cycles

## Electromagnetic emission

<b>Conducted RF emission</b>	
	The product must not send too much noise into the grid. This could lead to a disruption of other products.
<b>Test standard</b>	<b>Emission test class and group</b>
EN 55011	Class B, group 1

<b>Radiated RF emission</b>	
	The product must not emit RF signals in the room. This could lead to a disruption of other products.
<b>Test standard</b>	<b>Emission test class and group</b>
EN 55011	Class B, group 1

<b>Harmonic current emissions</b>	
	The product must not unintentionally draw power from the grid.
<b>Test standard</b>	<b>Emission test class and group</b>
IEC 61000-3-2	Class A

<b>Voltage changes, voltage fluctuations and flicker</b>	
	The product must not affect other products when it is connected to the grid. For example, these could be lights that could start flashing and thus bother people in the room.
<b>Test standard</b>	<b>Emission test class and group</b>
EN 61000-3-3	Tested according to the standard.

en

# Technical data

Materials	Powder laquered stainless steel Plastics (PVC, Polystyrene)
IP class	The electrical parts have an ingress protection class rating of IPX6. The product may not be immersed in water, but is designed to withstand powerful jets (12.5 mm nozzle) of water from any direction.

## Storage and transportation

- The temperature for transportation and storage of the unit must be between 5 and 40 degrees Celsius / 41 and 104 degrees Fahrenheit.
- The unit must always be transported in its original packaging.
- Avoid subjecting the packaging to shocks and impacts.
- Do not use the product if the packaging is broken or damaged.

en

# Disposal and recycling

The unit contains reusable materials. There are no known hazards or risks associated with disposal of the product. All components can be discarded after having been cleaned and disinfected.

When disposing of the product, we recommend that it be disassembled and broken down into different waste groups for recycling or combustion.

The following table provides information on the recycling and handling of the product.

<b>Recyclable components</b>	<b>Material</b>
Steel	Steel can be recycled by being re-melted and included as a secondary material in the production of new steel.
Plastics	Use plastic sorting and recycling systems where they are locally available, otherwise incinerating plastic has the added benefit of generating energy, which can be used for, for example, combined heat and power production.
Electronics	The electronic components should be collected separately and returned to the designated recycling service.

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