



*Access Your World*

## **Armedica Hi-Lo**

# **Changing Table User Manual**

**(Models AMSX1060/19855, AMSX1072/19855, AM150/19855, AM134/19855, AM140/19855)**

### **Important safety information**

**Do not leave occupant unattended.**

- Please read this manual thoroughly. We take no responsibility for any unspecified uses.
- The product is not intended to be played with.
- Do not use the changing table if it is out-of-order or damaged.
- This manual is subject to alteration without notice.
- Make sure that nothing interferes with the table when in motion.
- Children should not operate the changing table unsupervised.
- Verify voltage compatibility before initial operation.
- The table is not to be used in a shower.

### **Activating the table**

- Plug the power cord into a wall outlet. There should be no obstructions near or under the table.
- Apply the wheel brakes before transferring a person on or off of the table.
- Raise or lower the table by pressing the up or down button on the foot control to appropriate heights for wheelchair transfers and ergonomic working heights for caregivers.

Protect the vinyl with a waterproof cover-pad as needed during changing.

### **Safety Rails**

Raise or lower the safety rails by pulling sharply on the lock release handle. Both safety rails should be in the up position any time the table is occupied.

### **Moving the table**

Release the wheel brakes. The table is not intended for transporting an occupant. To prevent inadvertent damage, secure the foot control during transport.

Subject to change without further notice

[www.max-ability.com](http://www.max-ability.com) 800-577-1555 Fax: 707-575-3856 email: [Info@max-ability.com](mailto:Info@max-ability.com)

rev. Feb 2018



## Cleaning

Clean the vinyl after each use with a soft cloth with soap and water. Most general purpose cleaners and disinfectants can be used. Avoid harsh brushes, abrasives, undiluted bleach products, or cleaning agents containing acids or chloride. Rinse off with clear water and dry with a soft cloth. Clean after every use. Clean the underside of the table daily.

## General Maintenance

We recommend quarterly preventative maintenance:

- Check the operation of the product.
- Check all moving parts for wear.
- Check all fasteners, and tighten as necessary.
- Check the power cord and up/down control.
- Check vinyl for any damage and clean as recommended.
- Report of any defects, damage, or missing parts.

Lubrication and adjustment should not be necessary.

Verify your purchase details before carrying out repairs so you don't void your warranty:

*We guarantee all product components against manufacturing defects for five years, and parts repair/labor for one year, from the invoice date. In most instances the affected parts are replaced, not repaired.*

*Parts damaged due to improper product assembly, installation, or procedure; lack of recommended maintenance including cleaning; or lost after delivery cannot be included in this warranty.*

If you are unable to solve any problem, please contact MAX-Ability at 1 800 577-1555 or [www.max-ability.com](http://www.max-ability.com).