



*Access Your World*

# **Pressalit Care LS-FO Giraffe Daycare Changing Table User Manual**

**(Models R8671/R8672)**

## **Important safety information**

### **Do not leave occupant unattended.**

- Please read this manual thoroughly. We take no responsibility for any unspecified uses.
- The information in this manual pertains to correctly installed equipment.
- The product is not intended to be played with.
- Do not use the changing table if it is out-of-order or damaged.
- This manual is subject to alteration without notice.
- Make sure that nothing interferes with the table when in motion.
- Children should not operate the changing table unsupervised.
- Verify voltage compatibility before initial operation.

### **Using the changing table**

- To position the safety rail, release the lock and pull from the underside of the bench. Swing the rail up and towards you. It locks into position automatically.
- To fold the safety rail out of the way, release the lock and swing the rail towards the underside of the bench. Push the rail against the underside of the bench to lock into place.

### **Height adjustment/Folding**

- Adjust the height of the bench by using the indicators on the hand control. Press and hold the 'up' or 'down' button to initiate. Release to stop the movement.
- Table folds easily to the wall via integrated gas cylinders.

Subject to change without further notice



## Cleaning

All components are water compatible. Most general purpose cleaners and disinfectants can be used. Avoid harsh brushes, abrasives, undiluted bleach products, or cleaning agents containing acids or chloride. Rinse off with clear water and dry with a soft cloth. Clean after every use.

Occasionally, remove limescale marks on aluminium parts. Use 3 parts 30% household vinegar to 7 parts water. Rinse with a cloth wrung out in clean water.

Optional Mattress:

Upholstery Cover: Clean daily with soapy water, then wipe with a damp cloth. Wash once a week (white in warm water; red/blue in cold) without fabric softener. Do not turn inside out before washing. White can be tumble-dried and disinfected with antiseptic wipes. Do not use disinfectants intended for hard surfaces.

Foam Pad: Spot clean only. Do not machine-wash the foam. For hygiene reasons, do not use the mattress without the cover.

## Maintenance

Once a week move the product to its highest and lowest levels to lubricate the internal parts.

Inspect the changing table completely each year (with a licensed contractor if necessary):

- Remove the upper and lower securing screws of the coach housing and carefully remove the housing.
- Check all moving parts for wear.
- Check all mountings on changing table, safety rail and securing brackets and tighten as required.
- Check all electrical connections to the control box.
- Check isolation fuse (5 Amp) and change if incorrect.
- Test operation of all systems.
- Carefully refit the coach housing and secure.
- Report of any defects, damage, or missing parts.

Subject to change without further notice

[www.max-ability.com](http://www.max-ability.com) 800-577-1555 Fax: 707-575-3856 email: [Info@max-ability.com](mailto:Info@max-ability.com)

rev. Feb 2018



## Troubleshooting:

Verify your purchase details before carrying out repairs so you don't void your warranty:

*We guarantee all product components against manufacturing defects for five years, and parts repair/labor for one year, from the invoice date. In most instances the affected parts are replaced, not repaired.*

*Parts damaged due to improper product assembly, installation, or procedure; lack of recommended maintenance including cleaning; or lost after delivery cannot be included in this warranty.*

If the changing table stops functioning, check the power, wiring, and plugs. If there is something wrong or suspicious, please contact a qualified electrician or MAX-Ability for further assistance.

Resetting the changing table (in case of power interruption or table jamming):

1. Lower the changing table to its starting position. Release the 'down' button.
2. After 3-4 sec. press it again for approx 10 sec. until you hear a slight click; then release.
3. Raise the changing table to its highest position. You will hear another click.
4. Lower the changing table a couple of inches.
5. Raise it again to its highest position. You will hear another click.

The changing table is now reset.

For any questions or problems, please contact us at 1 800 577-1555 or [www.max-ability.com](http://www.max-ability.com).