



Access Your World

OmniMax Family Restroom Changing Table User Manual

(Models AM2572, AM3076)

Important safety information

Do not leave occupant unattended.

- Please read this manual thoroughly. We take no responsibility for any unspecified uses.
- The product is not intended to be played with.
- Do not use the changing table if it is out-of-order or damaged.
- This manual is subject to alteration without notice.
- Make sure that nothing interferes with the table when in motion.
- Children should not operate the changing table unsupervised.
- Verify voltage compatibility before initial operation.
- The table is not to be used in a shower.

Activating the table

- Plug the power cord into a wall outlet. There should be no obstructions near or under the table.
- Raise or lower the table by pressing the up/down toggle button at the end of the table to appropriate heights for wheelchair transfers and ergonomic working heights for caregivers.

Use waterproof cover-pads for extra protection as needed during changing.

Safety Rails

Raise or lower the safety rails by pulling sharply on the lock release handle. Both safety rails should be in the up position any time the table is occupied.

Subject to change without further notice

www.max-ability.com 800-577-1555 Fax: 707-575-3856 email: Info@max-ability.com

pub. Aug 2018



Cleaning

Clean after each use with any non-bleach over-the-counter general purpose cleaner and/or disinfectants. Avoid harsh brushes, abrasives, undiluted bleach products, or cleaning agents containing acids or chloride. Clean the underside (chassis) of the table daily. Do not use or clean in a shower environment.

General Maintenance

We recommend quarterly preventative maintenance:

- Check the operation of the product.
- Check all moving parts for wear and ease of movement.
- Check all fasteners, and tighten as necessary.
- Check the power cord and up/down control.
- Confirm proper function of safety belt and safety rails.
- Check the edge and underside of the table top.
- Report of any defects, damage, or missing parts.

Report any issues to the facility supervisor or MAX-Ability.

Lubrication and adjustment should not be necessary.

Verify your purchase details before carrying out repairs so you don't void your warranty:

We guarantee all product components against manufacturing defects for five years, and parts repair/labor for one year, from the invoice date. In most instances the affected parts are replaced, not repaired.

Parts damaged due to improper product assembly, installation, or procedure; lack of recommended maintenance including cleaning; or lost after delivery cannot be included in this warranty.

If you are unable to solve any problem, please contact MAX-Ability at 1 800 577-1555 or www.max-ability.com.