



Access Your World

Scalamobil X7 Scalaport

Universal Wheelchair Carrier

Beginner's Guide

Do not leave occupant unattended.

Review the Owner's Manual thoroughly before using the Scalamobil.

Practice attaching and removing the wheelchair.

Operate the Scalamobil without a passenger up and down the stairs first.

Be sure all operators have been thoroughly trained.

1. Pull the black knob to unfold the Scalaport.
2. Use the grey lever to fold the Scalaport.
3. To load the wheelchair and rider, back the wheelchair to the Scalamobil and adjust the wheel trays to match the width of the wheelchair wheels.
4. Using the motor, tilt the Scalamobil toward the wheelchair. This will make it easier to roll the wheelchair onto the Scalaport. Stand behind the Scalamobil and pull the wheelchair back so that the back wheels of the chair are in the trays. Apply the wheelchair brakes.



Beginner's Guide (cont.)

5. Attach the holding clamps to the upper part of the wheelchair vertical frame. The clamps are adjustable horizontally and vertically. Squeeze the clamps together on the wheelchair frame. Return the Scalamobil wheels to the neutral/lined-up position.
6. Apply the safety belt to the rider.
7. Reverse the process for unloading.
8. See the Scalamobil operator highlights and user manual for further instructions.

Call us at 800 577-1555 for support or assistance.

Subject to alteration without further notice.